# INSTRUCTIONS FOR USE (IFUs)

## NON-STERILE, NON-INVASIVE – ELITE CLAMPS

### INTENDED USE:

To occlude medical grade tubing 1 to 8 mm outside diameter.

### INSTRUCTIONS FOR USE:

(1) Make sure jaws grip tubing completely. (2) Press clamp together until tubing is fully occluded and the locking ratchet is engaged. (3) Observe clamp and tubing while in use and verify clamp stability and proper occlusion. Refer to illustration for safest occlusion.

### CLEANING INSTRUCTIONS:


(1) Use warm water to clean after use, rinse and dry. (2) Always refer to your facility standard cleaning protocols.

### WARNINGS AND CAUTIONS:

(1) Never freeze or expose clamp to temperatures over 130°C. (2) When cleaning clamp follow your own internal protocols to prevent over exposure to chemicals during the cleaning process. (3) When cleaning or sterilizing, it is suggested that the clamp is unlocked (open position). (4) The clamp will suffer weariness and discoloration from repeated use, cleaning and/or sterilization.

### LIMITED WARRANTY:

Avago Medical warrants that each clamp is free from defects in materials and workmanship. Repeated use, which requires cleaning and/or sterilization is the responsibility of the user.

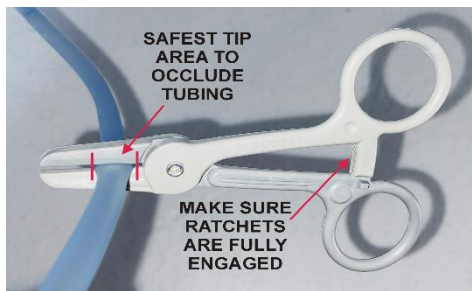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